

F*ACT*S*

FOAMING AUTOCLAVE CLEANING TREATMENT SYSTEM EFFICIENT AND QUICK FOAMING REVITALISING SOLUTION

F.A.C.T.S Will Remove All Rust, Spots, Stains And Corrosion, As Well As Scale, Alkaline And Mineral Deposits On The Inside Walls Of The Autoclave And From Stainless Steel Carts And Trays.

- Foaming spray remains on the walls longer, allowing for better penetration and more effective cleaning.
- It simultaneously eliminates all noxious and unpleasant odours.
- F.A.C.T.S is free rinsing and leaves no residue.
- Non-aerosol pump is environmentally safe.
- Repeat process as often as desired.
- Regular use of F.A.C.T.S. will prolong the life of your stainless steel equipment.

DIRECTIONS FOR USE:

For Autoclave And Washer Disinfector Cleaning

1. Autoclave walls must be cool.
2. Spray F.A.C.T.S. generously on inside of the chamber.
3. Allow to remain wet for 15-30 minutes.
4. Brushing with non-metallic brush will speed the action.
5. Use of gloves is recommended.
6. Rinse walls with copious amounts of water.

For Stainless Steel Carts, Trays And Surfaces

1. Spray F.A.C.T.S. generously on surfaces.
2. Allow to remain wet for 15-30 minutes.
3. Brushing with non-metallic brush will speed the action.
4. Use of gloves is recommended.
5. Rinse the surface with copious amounts of water.



CAUTION: DO NOT USE ON CHROME OR SILVER PLATED EQUIPMENT

Available in:

Cat No 345FCT 650ml Spray Bottle

R **RUHOF** 
THE EXPERTS IN A BOTTLE

Murray Surgical Ltd
23a Moyle Road
Dublin Industrial Estate, Glasnevin, Dublin 11
Tel: 01 8308266 Fax: 01 8308129

E Mail: info@murraysurgical.ie Web: www.murraysurgical.ie