

Sink Auto Dispenser with Unique Wave Switch Touch-less Technology

An infection control feature that helps prevent contamination of the dispenser pump from soiled gloves

- Wave Switch Touch-less Technology
- Built in temperature display
- Temperature too high warning
- Dose reading and countdown
- Unit is splash proof and washable
- Unit is low voltage for safety



Benefits

- Touch-less technology, infection control benefit
- Validates manual cleaning processes
- Reduces chemical handling
- Risk of spillage removed
- Reduces chemical wastage
- Time saving
- Prevents under dosing

- These units can be programmed to put in as little or as much detergent as the customer requires consistently.
- Gives the user control of the chemical dosage and ensures cleaning quality consistency.
- By fitting the Auto dispenser the customer is in control of the wash quality, allowing the introduction of quality procedures for pre-soaking, manual washing and ultrasonic cleaning.



**Peskett
Solutions
Ltd**

Sink Auto Dispenser with Unique Wave Switch Touch-less Technology

An infection control feature that helps prevent contamination of the dispenser pump from soiled gloves

Installation

- Our office, will contact the customer to organise a convenient fitting time and date. The customer is responsible for giving the engineer the fill level details, to enable the engineer to set the pump with the correct dilution rate.
- Before the dispenser can be fitted, the customer will need to calculate the quantity of water they require in the sink to clean their surgical instruments. Usually, this level is determined by the level at which the instruments are fully submerged just below the water line. When the level has been agreed a permanent fill line needs to be established. This can be marked either with waterproof tape (supplied), a permanent marker or engraved into a stainless steel sink. Using the information supplied by the customer, the engineer can calculate the required dosage to set the pump. e.g. 30 Litres of Water = 120ml of Endozime AW Plus @ 4ml per Litre
- Personalised laminated instructions are supplied with each dispenser pump for the customer to put on the wall near the unit.

Calibration and Maintenance

- The auto dispensers are fitted and maintained by an approved independent dispenser engineer. The Dispenser pump is serviced and calibrated annually. The calibration certificate will assist your quality audit. The pumps require only a small amount of maintenance if used correctly.

Terms and Conditions

- The dispenser remains the property of "Pesckett Solutions Ltd", as the dispenser pump is supplied, fitted and maintained free of charge. It is the customers responsibility to ensure that the dispenser pump is operating correctly and if the dispenser pump is damaged by misuse, the customer will pay for the repair or replacement of the unit. If the customer ceases to use our product, the pump is removed by the engineer during the annual calibration.



**Pesckett
Solutions
Ltd**

Unit 4-5 Brampton Business Park
55 Brampton Road, Eastbourne
East Sussex BN22 9AF

Tel: 01323 511038 Fax: 01323 503247

Email: support@peskettsolutions.com Web: www.peskettsolutions.com